## SEDILE WC AUTOSOLLEVANTE **WC-BRITESEAT** SELF-LIFTING WC SEAT

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## **CARATTERISTICHE**

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Facile da installare.

Meccanico.

Affidabile.

Durevole.

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**Telefono: +39 059 283928  
email: mecoild@yahoo.it**

## **FEATURES**

Fits conventional WC bowls.  
Easy to install.  
Mechanical.  
Reliable.  
Durable.  
Easy to clean and to maintain.  
Supplied with stainless steel fixing bolts and instructions.


## **INTENDED USE**

Mec-Oild WC-BRITESEAT is a mechanical self-lifting WC seat designed to fit conventional WC bowls.

## **SAFETY INSTRUCTIONS**


Read all of this manual carefully before using the seat.  
The intended use is described in this manual. Any other use or the use of any accessory not recommended in this manual may present a risk of personal injury.  
Retain this manual for future reference.  
The seat is not intended for use by children.  
The seat is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision by a person responsible for their safety.  
The seat is not to be used as a toy.  
When not in use, the seat must be stored in a dry, well ventilated place out of the reach of children.  
Before use, check the seat for damaged or defective parts.  
Check for any other conditions that may affect the operation of the seat.  
Do not use the seat and uninstall it if any part is damaged or defective or if any other conditions affect the operation of the seat.  
Have any damaged or defective parts repaired or replaced by an authorised repair agent.  
Never attempt to remove or replace any parts other than those specified in this manual.  
Use original consumables and replacement parts only.  
The seat allows a maximum user weight of 120 kg.

**WC BOWL REQUIREMENTS**

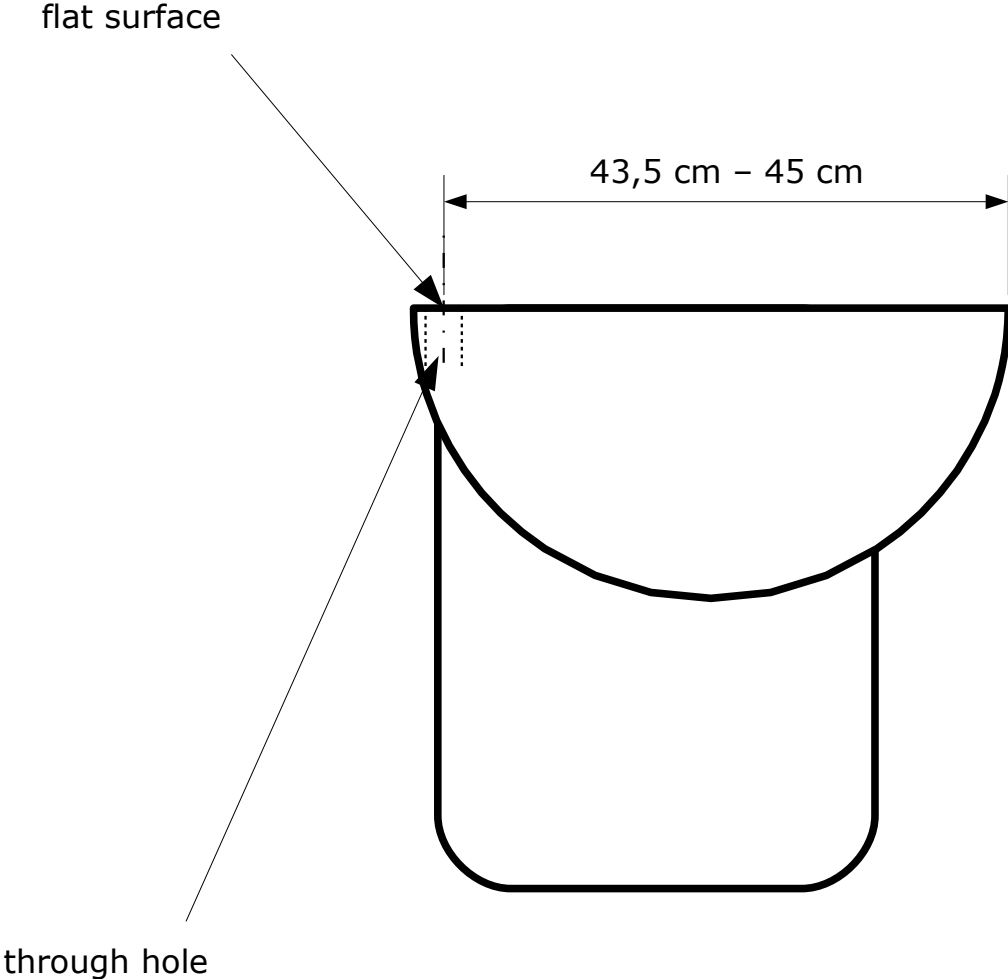**INSTALLATION**
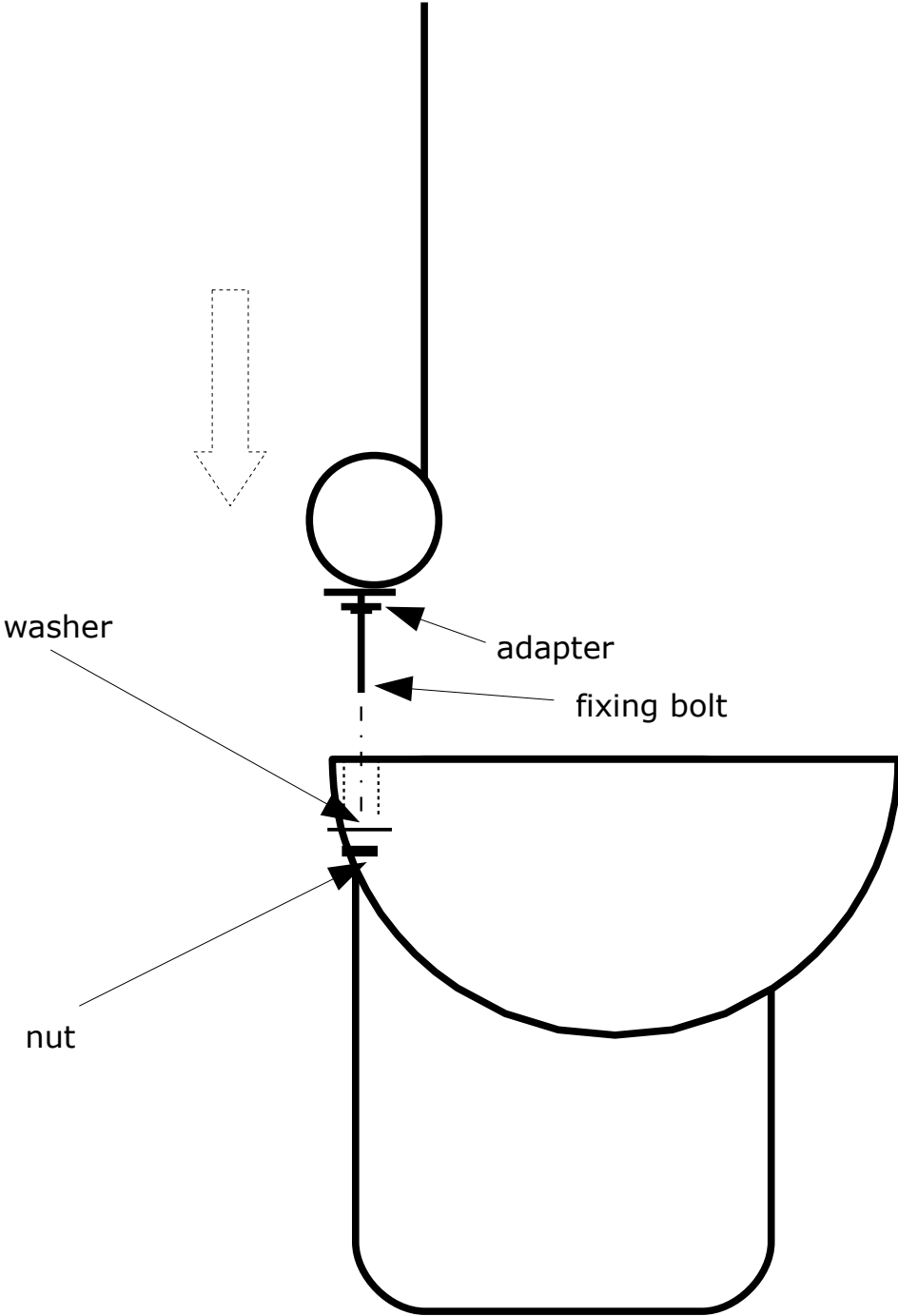
## **GUARANTEE**

This guarantee statement is in addition to and in no way prejudices your statutory rights. The guarantee is valid within the territories of the Member States of the European Union and the European Free Trade Area (EFTA).

If a Mec-Oild product becomes defective due to faulty materials, workmanship or lack of conformity, within 24 months from the date of purchase, Mec-Oild guarantees to replace defective parts, repair or replace the product unless:

- The product has been subjected to misuse.
- The product has sustained damage through foreign objects, substances or accidents.
- Repairs have been attempted by persons other than authorised repair agents or Mec-Oild service staff.

To claim on the guarantee, you will need to submit proof of purchase to the seller or an authorised repair agent.

**SERVICE**

**MEC-OILD  
Via Folloni 65  
41126 Modena (MO)  
ITALY**

**Phone: +39 059 283928  
email: mecoild@yahoo.it**

M0569